


JUVAPEN




Quick start guide

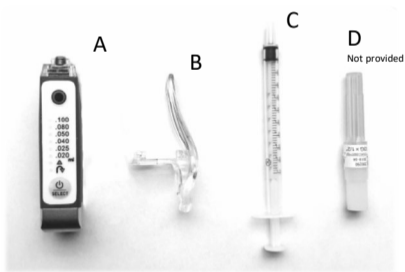

Activation button



Dose in millimeter

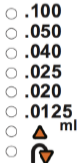

Continuous flow
Rewind mode




On / Off,
Select button




1

Fill syringe to desired volume

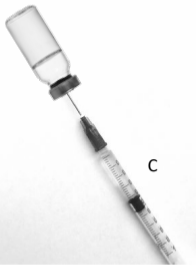


2

Remove air bubbles by flicking syringe

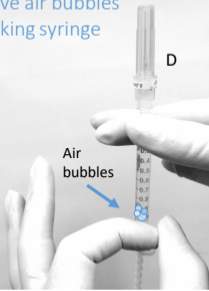


3

Break the plunger rod (breakable at 0.1ml increments)

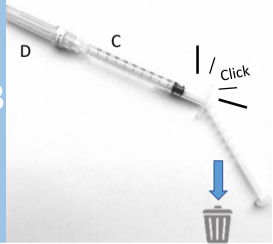


4

Attach the syringe holder

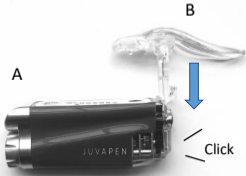


5

Attach the syringe

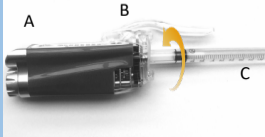

6

Turn on the Juvapen

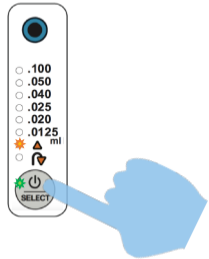

7

Select *Continuous flow* and press lever...



8

... to remove air and prime needle




Air bubbles




30°

All air bubbles **must** be removed to avoid product wastage.

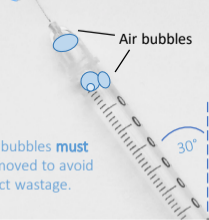

9

Select desired dose

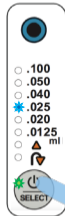


- .100
- .050
- .040
- ★ .025
- .020
- .0125 ml
- ▲
- ↻



1
0

Press and release lever
to deliver dose

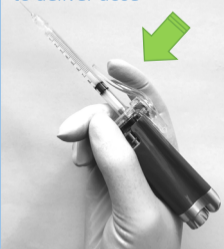

I N J E C T I O N S Y S T E M S

J U V A +

Visit juvaplus.com
for video instructions

P2.18R4